

# Neutra Skin<sup>®</sup>

## Pectin-Based Skin Barrier whenever Skin Needs Protection from Adhesives



Ultra-Thin Barrier

Economical

Developed for Neonatal Skin

Protection from Adhesives

Inexpensive to Use

**Neutra Skin** represents a major advancement in pectin-based skin barriers.

**Neutra Skin** has been developed to produce the lowest possible skin irritation while maintaining superior adhesion.

These properties are particularly important when the product is used in neonates.

**Product Number: 7OS2432, 15 sq. in. Square (5 squares per box)**

### Instructions for Use

1. Cut the **Neutra Skin** barrier to the desired shape.
2. Warm the skin barrier between clasped hands and then remove the protective cover from the skin barrier.
3. Carefully apply the barrier to the skin by maintaining gentle pressure.

**Distributor:**



Vital Care Products, Inc. • 3046 Brecksville Road, Suite 2; Richfield, OH 44286  
Phone: 800-837-5511 – e-mail: [sales@vitalcareproducts.com](mailto:sales@vitalcareproducts.com)